


# Overview to choose LARYVOX® Adhesive Base Plates

All LARYVOX® Base Plates are designed to carry an HME (Heat and Moisture Exchanger) filter cassette, e.g. the #49800 LARYVOX® HME.

However, the *wide array* of LARYVOX® Base Plates can make it hard to decide upon which one to choose.

Here is a simplified overview on how to choose:

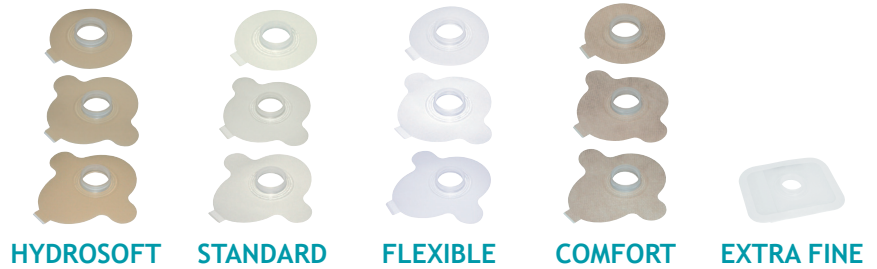

Question	HYDROSOFT	STANDARD	FLEXIBLE	COMFORT	EXTRA FINE
Have you recently had surgery or radio therapy?	😊	😊	😊	😊	😊
Do you have sensitive skin?	😊	😊	😊	😊	😊
Are you using a Voice Prosthesis?	😊	😊	😊	😊	😊
Are you speaking Hands Free?	😊	😊	😊	😊	😊
Is your stoma deep or irregular?	😊	😊	😊	😊	😊
Are you using Adhesive Base Plates in the shower?	😞	😊	😊	😊	😊



### 1. HYDROSOFT

Especially suited for use directly post-op, or on sensitive/damaged skin. Does not stick as well as the other adhesives and should not be used in the shower.

### 2. STANDARD

Less sticky, but soft and very well suited for those without a voice prosthesis.

### 3. FLEXIBLE

Especially suitable for the typical user using mostly standard HME filter cassettes that are finger occluded. Can work very well for Hands Free speakers especially in combination with a silicone glue to enhance the seal.

### 4. COMFORT

The latest generation of adhesive suiting those using finger occluded HME cassettes as well as Hands Free speakers. More comfortable, longer lasting. Silicone glue might not be necessary.

### 5. EXTRA FINE

Unique adhesive properties; extremely flimsy which helps reaching and filling every wrinkle in deep and/or irregular stomas. Water repellent and the longest durability. The extended duration could however wear on the skin – but properly placed it might last a week or even longer.

## How to get the LARYVOX® Adhesive Base Plates to last longer

1. When removing a Base Plate, use OPTICLEAR® Adhesive Remover (SKU 33500). It contains a glue-dissolving solution. Apply it onto the adhesive, and then work on the skin while lifting up and removing the Base Plate. This solution leaves an oily residue on the skin that has to be cleaned before applying a new Base Plate.
2. Clean the skin with OPTIFAHL® Stoma Cleaning Wipes (SKU 33200), or soap and water. Let dry.
3. Treat the skin with OPTIGARD® Skin Protection Film (SKU 33600). It can help prevent skin damage from using adhesives. Let dry.
4. Place the Base Plate in your hands and warm it up.
5. Apply the Base Plate by ensuring tight seal at the 6 o'clock position of the stoma opening. Then 'tilt up' the rest of the Base Plate in place.
6. Ensure a good tight fit by massaging onto the Base Plate, and make sure all wrinkles are stretched and filled out.
7. Wait 10-15 minutes before speaking, or 30 minutes if you use a Hands Free device.

*Talk to your SLP on how to speak without effort  
– the less effort you use, the longer the Base Plate will last!*